


NEW

Any-Cem[®] Implant

Dual-cure resin cement for stable and aesthetic temporary cementation of Implant Crowns

Strong Retention, Easy Removal

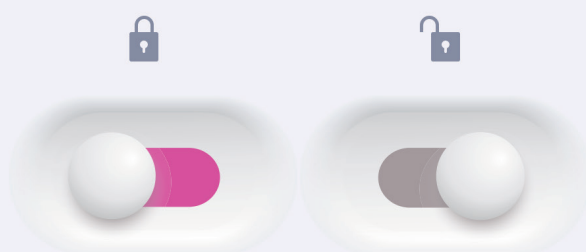

MDCLUS
MEDICLUS CO., LTD.

Any-Cem[®] Implant

A dedicated temporary cement designed for strong retention and damage-free removal of implant crowns and various prostheses.

Any-Cem Implant has been specially developed for the strong and reliable temporary cementation of implant crowns, bridges, inlays, onlays, and veneers. Its elasticity responds flexibly to masticatory forces, absorbs shocks, and provides a tight seal at the margins and gaps. When needed, it allows for the comfortable and easy removal of prostheses. As a eugenol-free, dual-cure cement, this product does not interfere with the final bonding of permanent restorations. Available in both White and Pink shades, it is also ideal for the aesthetic temporary cementation of anterior restorations.

Confidence for the clinician, comfort for the patient — strong and stable temporary fixation you can trust.


Indications

- Temporary cementation of implant-retained crowns, bridges, inlays, onlays, and veneers
- Temporary cementation of all-ceramic and composite resin restorations


Benefits

- Secure Retention, Effortless Removal
- Adequate strength, resilience, and elasticity: Special UDMA component
- Sufficient working time: 2min 30sec (25°C), 1min 30sec (37°C, Self-cure)
- Two polymerization options enhancing efficiency: Light-cure, Self-cure
- Easy excess removal after tack-curing
- Two aesthetic shades to match clinical needs: White, Pink
- Convenient application with Automix Double Push syringe

Step by Step Process

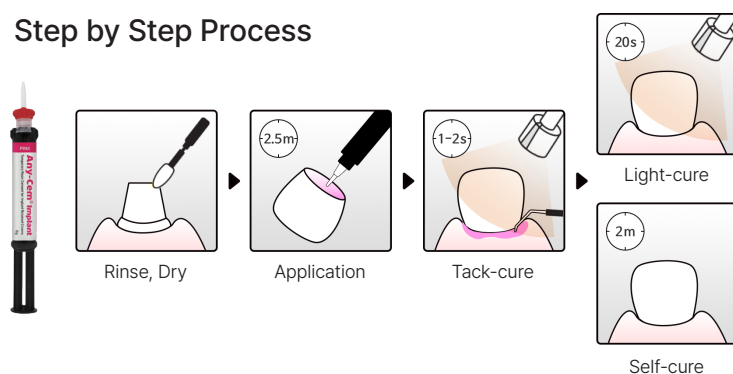

Radiopacity

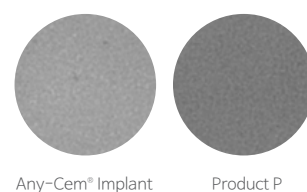

* Source: Dr. SG Cho, Luna conservative dental clinic

Delivery Forms



#202-1 Any-Cem[®] Implant Refill 6g 1ea, Pink

Automix tips 10ea
Eco-tip 1ea
Mixing pad 1ea

#202-2 Any-Cem[®] Implant Refill 6g 1ea, White

Automix tips 10ea
Eco-tip 1ea
Mixing pad 1ea

MEDICLUS

Headquarter | No.1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si, Chungcheongbuk-do, 28576, Korea | +82 43 211 2877

Sales Office | No.206 Brown Stone, 464, Cheongpa-ro, Jung-gu, Seoul, 04510, Korea | +82 10 8385 2877

www.mediclus.co.kr | sales@mdclus.com

