

Protect Completely, Remove in One Piece

Advantages

✓ **Precise Application**

Optimized viscosity enables accurate placement along the gingival margin without unwanted flow.

✓ **Reliable Protection**

Forms a uniform and durable barrier that effectively protects soft tissue from chemical irritation.

✓ **Residue-Free Removal**

Easily peels off in one piece, leaving no residue behind.

✓ **Bluish Color for Enhanced Visibility**

Provides excellent visual contrast for precise control, allowing easy identification and complete removal of the material, thereby improving clinical efficiency.



Dentto-Dam™

Light-cured liquid dam for precise isolation



Dentto-Dam™

✓ Protect Completely, Remove in One Piece

Dentto-Dam™ is Gingival Protection and Rubber Dam Sealing for Precise and Convenient Treatment. It is a light-cure liquid dam designed to gently protect gingival tissue and enhance sealing during rubber dam placement, helping clinicians achieve precise and efficient treatment.

✓ Indications

- Gingival protection during whitening treatment
- Gingival barrier against etchant, blasting powder, and other irritants
- Auxiliary sealing with rubber dam application
- Temporary wall formation during endodontic treatment

✓ Clinical Photos




Whitening (Wash, dry)




Apply & Light curing (20s)



Remove (in one piece)

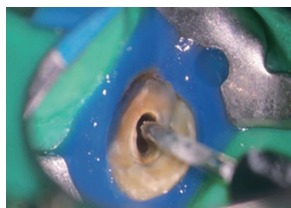

Isolation Rubber Dam I

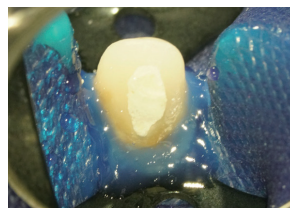

Isolation Rubber Dam II



Endo-Wall

✓ Application Procedure (For Whitening Treatment)



① Clean and dry the teeth



② Apply the dam



③ Light cure (10s)



④ Remove the dam

CONTENTS

#402 Dentto-Dam™ 1.2 ml x 5 ea,
Disposable tips



Seoul Office +82-10-8535-2877

Head Office Research & development center

No. 206, 464, Cheongpa-ro, Jung-gu, Seoul 04510, Korea
Fax : +82-2-6499-1006 E-mail : sales@mdclus.com

No. 1210, 134, Gongdan-ro, Heungdeok-gu, Cheongju-si,
Chungcheongbuk-do 28576, Korea

MDCLUS
MEDICLUS CO., LTD.