

Any-Cem®

SELF-ADHESIVE RESIN CEMENT

Any-Cem is a dual cured, self-adhesive resin cement that requires no pre-treatment of tooth substructure. It shows high bond strength to various prostheses, such as zirconia, metal, composite resin, and glass ceramic.

Advantages

- Conveniently and quickly remove excess cement with tack-cure
- High bond strength
- Dual-cure
- No pretreatment required




Physical properties


	Any-Cem	Product B
Working Time	3'30"	2'00"
Setting Time	4'30"	6'00"

EASY CLEAN-UP
EASY HANDLING


Bond strength

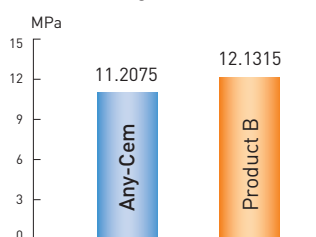

Bending strength

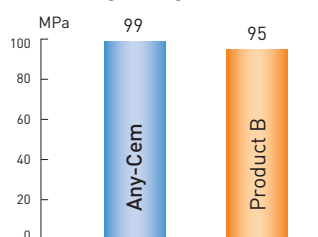

Film thickness

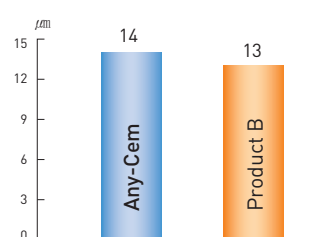

How to use




- Remove the cap of Any-Cem



- Close the tip tightly in front of the syringe



- Apply Any-Cem inside crowns or bridges

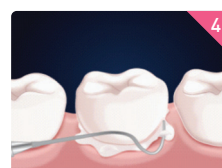

- Set the applied crowns or bridges on the teeth and safely remove the excess cement before the final set



- Polish the margin and check the occlusion



SELF-ADHESIVE RESIN CEMENT

CONTENTS

203 Any-Cem 6g x 1 ea

203-1 Any-Cem White 6g x 1 ea

203-3 Any-Cem TL 6g x 1 ea